

User manual

CPR Cat - *CaPpeR*

Item no.: VET4880



*Experts in medical education
since 1950*



ERLER ZIMMER

Erl-Zimmer GmbH & Co. KG · Hauptstr. 27 · D-77886 Lauf
Tel.: (+49) 07841/67191-0 · info@erler-zimmer.de · www.erler-zimmer.de

Use the following tool supplied to insert the lung bags into the model.



Press the heat-shrink tubing and the cat's nostrils together.



Pass the tube through the cat's nostrils until it appears in the cavity.



Feed the wire with the narrow, angled side through the heat-shrink tubing.



Thread the lung bag, open side first, into the constricted area of the drath and pull it back through the shrink tubing.



Now spread the lung bag over the nose.



Spread out the lung bag in the cavity.



Place the foam block in the cavity.



Lay the cat on its flat side with the foam block facing the floor.



The simulator is now ready for use.



Erler-Zimmer GmbH & Co.KG

Hauptstraße 27

D-77886 Lauf

Tel.: +49 (0)7841 / 6003-0

Fax: +49 (0)7841 / 6003-20

E-Mail: info@erler-zimmer.de